

SEPTEMBER 2022 MONTHLY HBOT WEBINAR

Clean Chambers are Safe Chambers

PRESENTED BY THE VIA
CHRISTI PROGRAM




www.serenagroupinc.com



Protect their Longevity



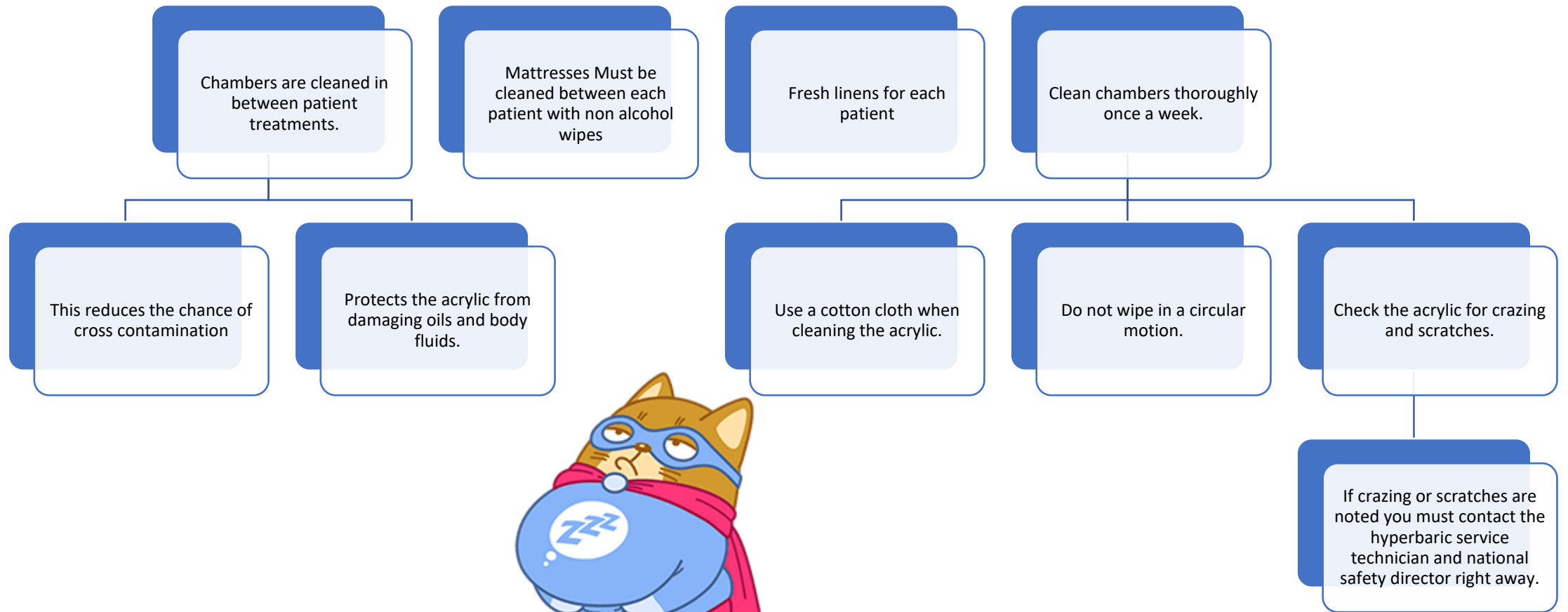

Cleaning

- The muffler should be removed from the chamber weekly and cleaned with a wire brush and soapy water.
- Dust the chamber and surrounding areas. Keep the area clean and free of dust. The use of a swifter or other duster is recommended.
- If accumulation of dust is notice in the door lock of the chamber attempt to clean the dust away. If the integrity of the door lock becomes compromised contact the HBO service technician.



Benefits of Maintaining the HBO chambers

- Protects the longevity of the chamber
- Health benefits of our patients.
- Reduces contact infections such as MRSA

NOTE

- Hand sanitizers can be utilized in the hyperbaric center, BUT the hand sanitizer cannot have a high percentage of alcohol-based products. Should not be used before going into the chamber.
- Therefore, it is not recommended to have alcohol-based hand sanitizers in the hyperbaric chamber room.



Approved cleaning



CARE AND DISINFECTION OF THE CHAMBER AND GURNEY


- Clean the chamber according to the type of cases being treated and as directed by the medical staff. Refer to the list of acceptable cleaners available from SECHRIST INDUSTRIES.
- Wash Stretcher, Pillows, Positioning Pads and Mattress surfaces with approved cleaner or mild dish washing soap. Refer to the Gurney User Manual for cleaning instructions.

CLEANING OF PAINTED AND ANODIZED SURFACES

- Use the same methods for cleaning as used for the Acrylic Cylinder.

APPROVED CHAMBER CLEANERS

DISINFECTANT	MANUFACTURER CITY, STATE	MANUFACTURER PART NUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDED DILUTION
Asepti-HB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	Not Provided	Ready to use	N/A
Vinsept	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6002314	Not Provided	Ready to use	N/A
Oxyde Daily Disinfectant	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6000189	1:128-1:64	1:128-1:64	YES
Neutral Disinfectant Cleaner	Ecolab Professional Products Ecolab Inc. St. Paul, MN	6027314	1:256-2:64	1:256-2:64	YES
Tic-HB (Discontinued, can use unexpired stock)	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:64	1:64	YES
Sant III TB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	1:128	YES
Bemcoup	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	1:128	YES
LpH-se	Calgon, Vestal Division STERIS Corp. Mentor, OH	N/A	1:256	1:256	YES
Sani-Cloth Bleach Wipe	FDI/Neopak Orangeburg, NY	U13295	Equivalent to 1:10 dilution	Ready to use	N/A
Clorox Healthcare Bleach Germicidal Cleaner	Clorox Professional Products (Oakland, CA)	68967, 68970, 68832, 68973, 68978	Not provided	Ready to use	N/A
Dispatch with Bleach	Caltech Industries, Inc. Midland, MI	68832	Not provided	Ready to use	N/A
Quaternary Disinfectant Cleaner	ECOLAB® St. Paul, MN	6063301 6063304	1:256-1:64 Not provided	1:256 - 1:64 Ready to use	YES N/A
Covage® Spray HB Plus	Stein Corporation Mentor, OH	1624-77	Not provided	Ready to use	N/A
Sani-Cloths® HB	FDI/Neopak Orangeburg, NY	Q08472	Not Provided	Ready to use	N/A

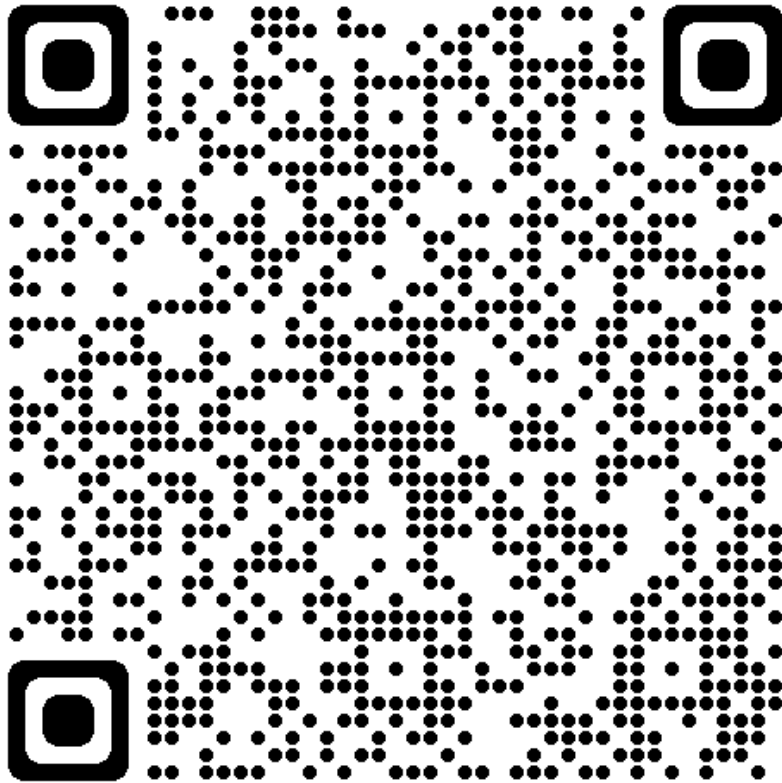

Websites

- Perrybaromedical.com
- Sechristusa.com



QUIN

Question 1

- If the first patient of the day has MRSA, it is adequate to clean the chamber at the end of the day.
- True
- Or
- False



Answer 1

- The answer is

FALSE

Question 2

- Where do you find the list of approved chamber cleaners?



Answer 2

- Perrybaromedical.com
- Sechristusa.com



Question 3

- How often should the chambers be cleaned?



Answer 3

- Between each patient
- Weekly

Question 4

- Alcohol-based cleaners are best for cleaning the mattress.

- True
- Or
- False



Answer 4

- FALSE,
- NON ALCOHOL WIPES OR CLEANERS SHOULD BE USED.

Question 5

- What cloth is best to be used to wipe the acrylic?
 - a. Paper towels
 - b. Microfiber towels
 - c. Cotton fabric

Answer 5

- Cotton fabric



[This Photo](#) by Unknown Author is licensed under [CC BY](#)

SerenaGroup HBOT Contact Information

Matt Schweyer, Chief Compliance Officer | VP Head of Compliance, Reimbursement & Safety | National Safety Director

- mschweyer@serenagroups.com
- C. 214-315-5109

SerenaGroup Education Committee

- Ally George = HBO Educator
 - ageorge@serenagroups.com
 - C. 609-202-6152
- Blair Flinn
- Nancy Trafelet
- Jill Schroder



SerenaGroup Upcoming HBOT Educational Courses

LIST DATES AND LOCATIONS

- November 3-6, Wichita, KS



Next Month's Presenter

DATE: October 18, 2022 at 12 pm
eastern time

PRESENTING: Inspira Health

TOPIC: Physiology of HBOT for
Each Indication and When to
Consider Continuation



Changes

- Daily Checklist
- Weekly Checklist
- Dive Log

Round Table Discussion

